


## Attaches to:

ECO2, ECOpoint (with eyegaze),  
ECO-14, Vanguard, Pathfinder 2 and Accent 1400


Device to  
device plate

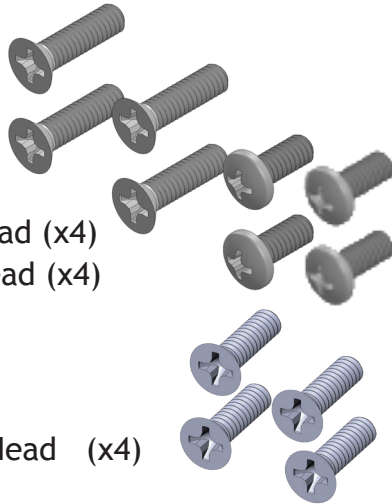


8-32 x 5/8" flathead (x4)  
8-32 x 3/8" panhead (x4)

6/32 x 1/2" Flat Head (x4)

Quick release  
plate (QRP) to  
device plate

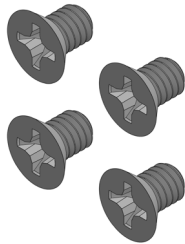

8-32 x 1/4" (x4)

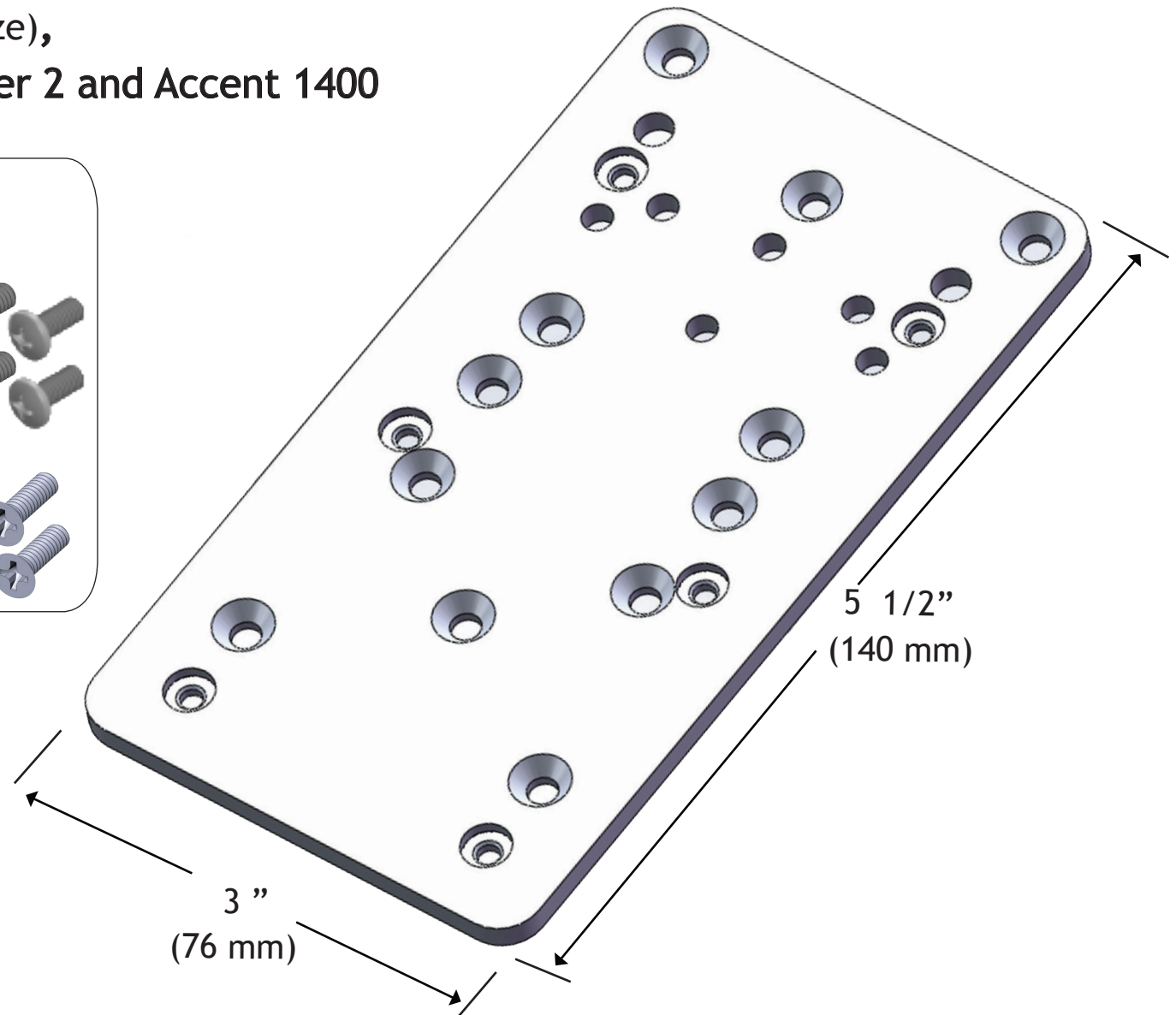

**Mount'n Mover™**  
by BlueSky Designs

[mountnmover.com](http://mountnmover.com)


**Product:** DP-PRC1, Prentke Romich Co.  
**Weight:** 3.8 oz. (0.11 kg)

1-888-724-7002 (toll free) or 1-612-724-7002

pg 1 of 3

## ECO 2 and ECOpoint (with eyegaze)


\*For all models, the original mounting plate screws should be removed first.

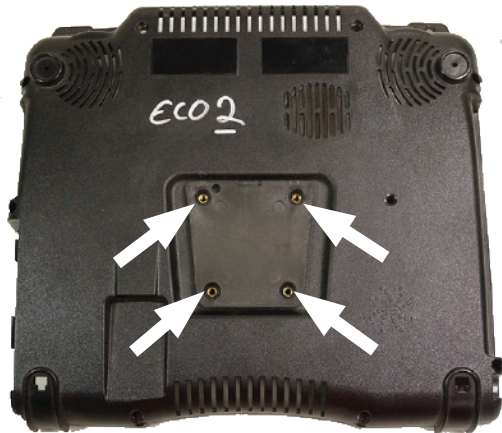


Leave  
mounting  
plate in  
place



Attach  
device  
plate

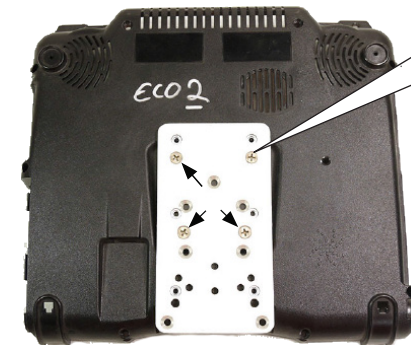

Mount QRP lower  
when attaching to  
the **ECOpoint** (eye gaze)

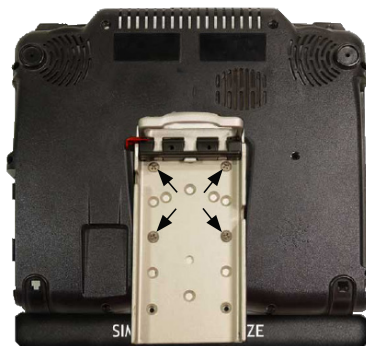


ECOpoint

(Attach before QRP)

Stand 90  
or  
Universal  
Rotator

ECO2

Attach  
QRP to  
**ECO2**

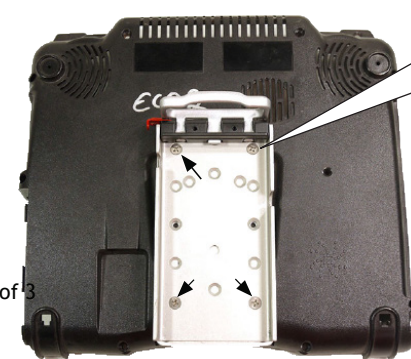


pg 2 of 3




# PRC Accent 1400

①

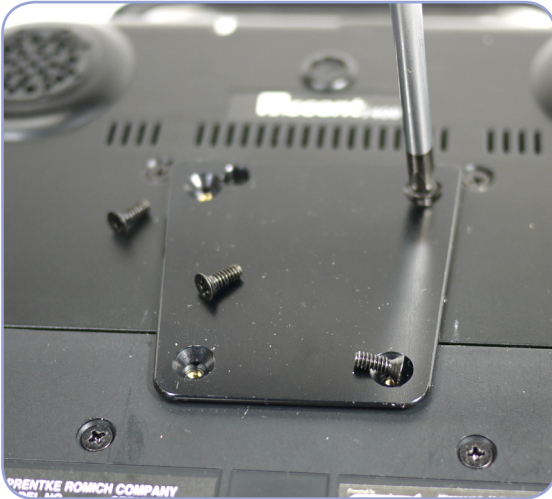


Remove screws from mounting bracket.

②

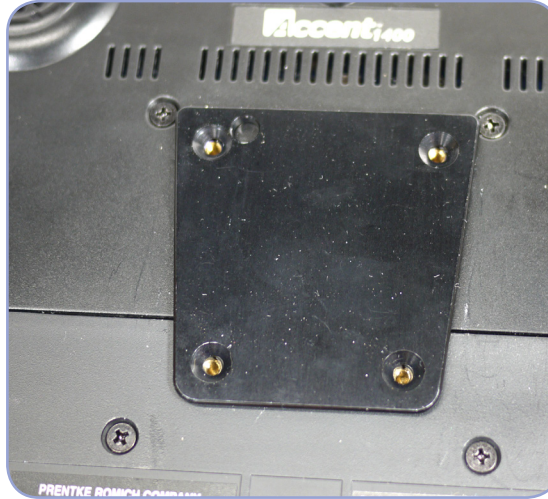

Leave mounting bracket in place.

③

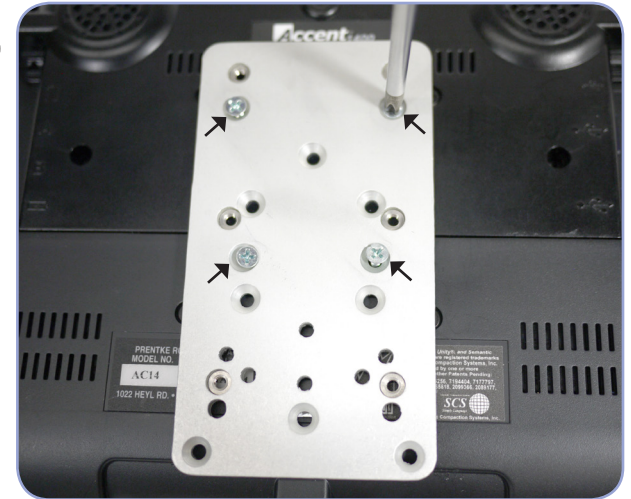

Place DP-PRC1 over the Accent 1400 mounting bracket. Line up the four holes between the two plates and the Accent 1400. Attach using the four screws provided (6/32 x 1/2").


④



Attach the Quick Release Plate to the DP-PRC1 using the four screws provided (8/32 x 1/4")

④

cont.

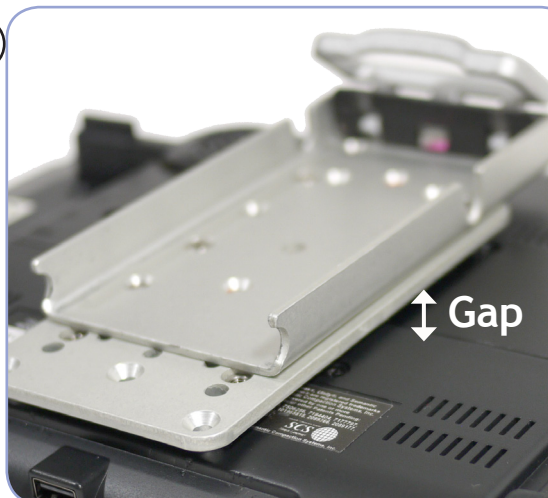

There should be a gap between the DP-PRC1 plate and the Accent 1400 to allow for ventilation.

⑤



Attach the Quick Release Plate (QRP) to the Mount'n Mover!