

Attaches to:

Tobii C8, C12, CEye (eyegaze),
P10, LightWriter SL40, S32

Device (C8,
C12, CEye)
to device
plate




M4-.7 x 25mm (x4)
1/4" x 11/16" spacer, aluminum (x4)

Device (P10,
S32, and
Lightwriter) to
device plate




M4-.7 x 10mm (x4)

Quick release
plate (QRP) to
device plate

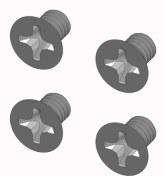

8-32 x 1/4" (x4)

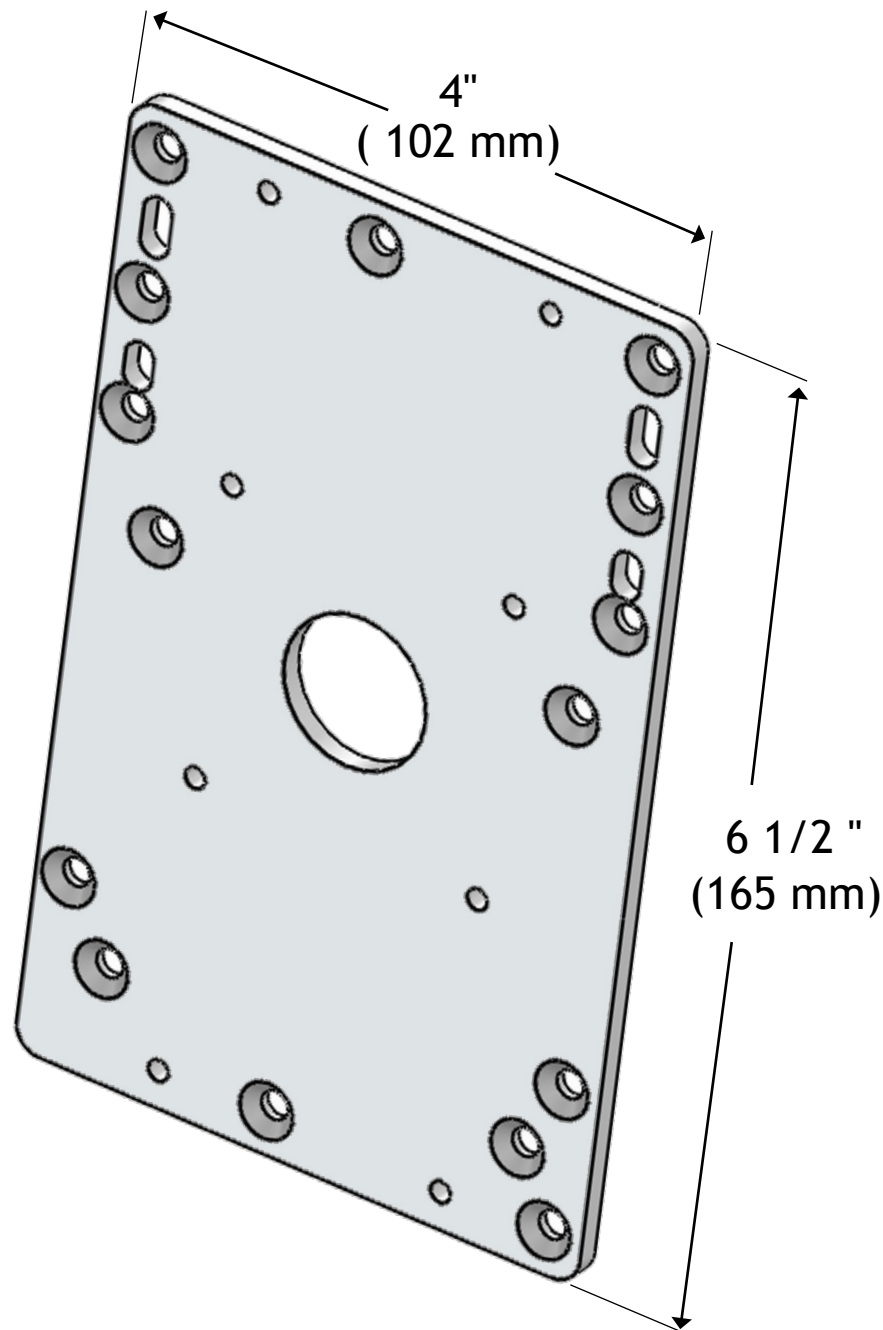

Mount'n Mover™
by BlueSky Designs

mountnmover.com

Product: DP-T1, Tobii
Weight: 5.8 oz. (0.16 kg)

1-888-724-7002 (toll free) or 1-612-724-7002


pg 1 of 5

C8



Remove plate and "kickstand" on both C8 and C12


C12

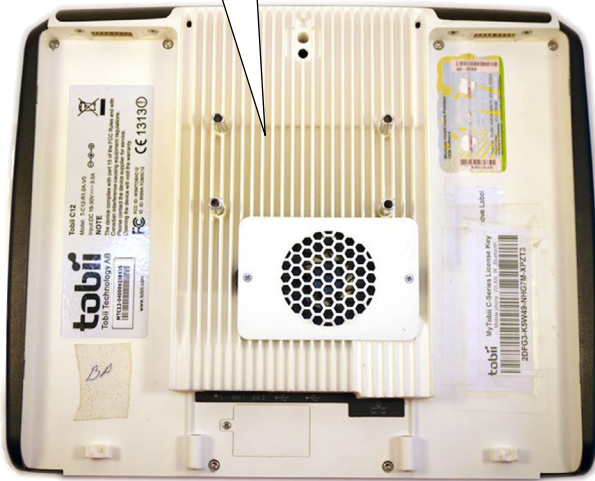

(C12 with plate and "kickstand" removed)

Place Spacers


C8 and C12

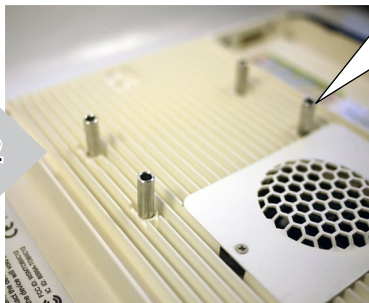

x4
1/4" x 11/16"


Attach device plate (DP-T1) over spacers on C8 and C12

C8

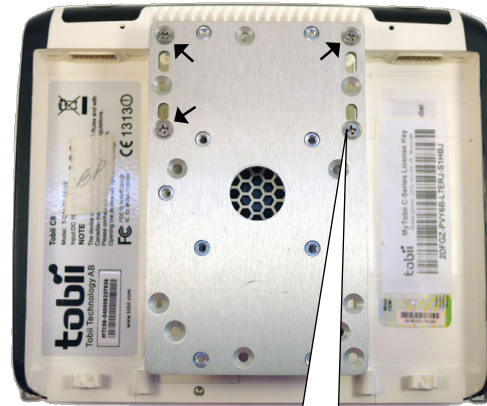


x4
M4-.7 x 25mm

C12

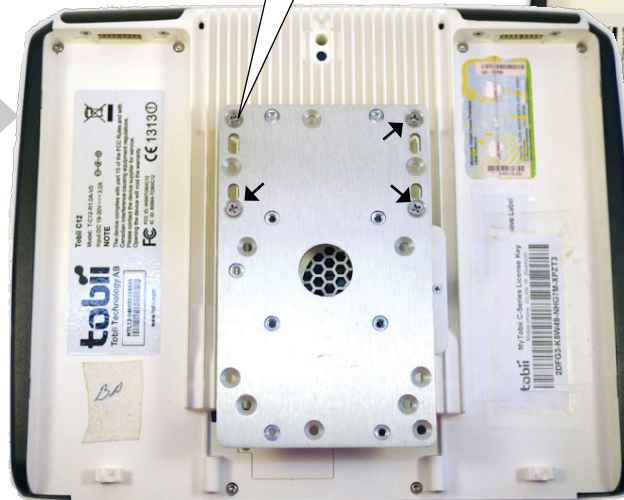


Attach QRP

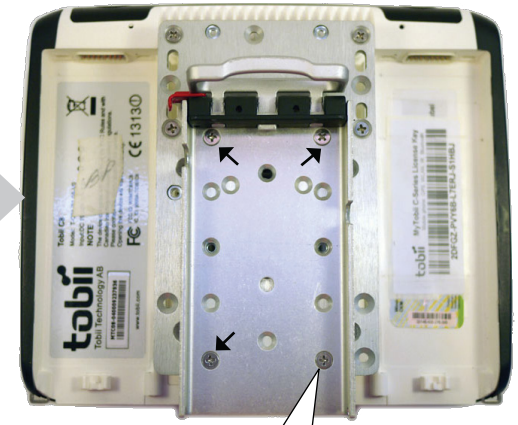


x4
8-32 x 1/4"

Attach QRP

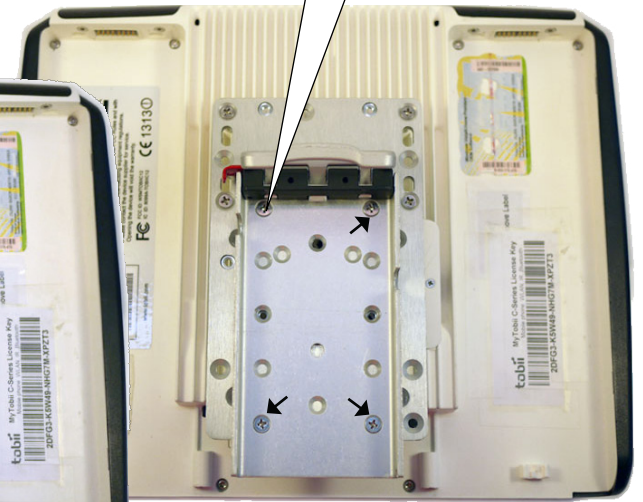




C12 with cEye (eyegaze)

*Attachment option without using spacers

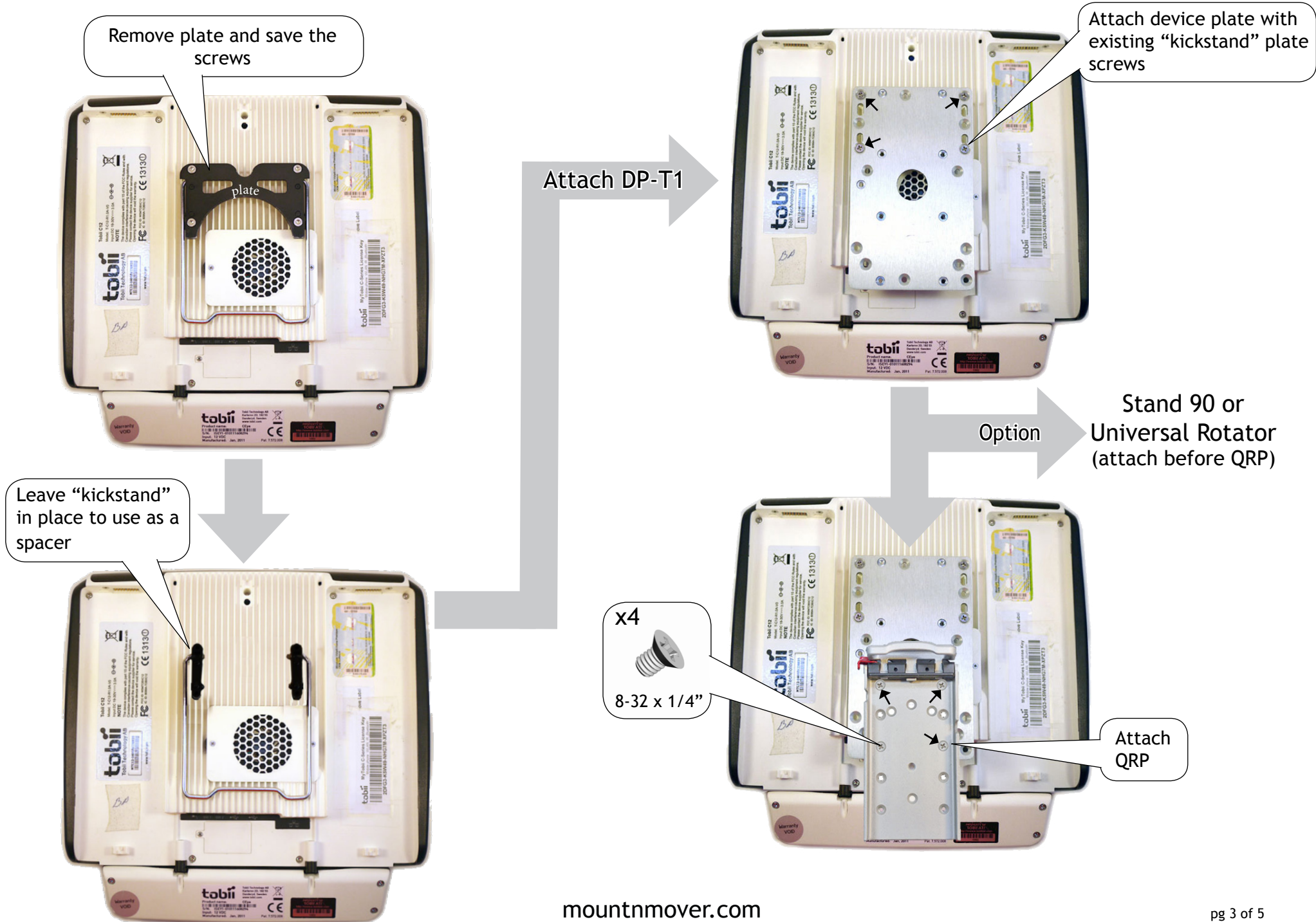

P10

✱ Remove any mounting hardware that may be attached to the device.

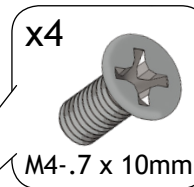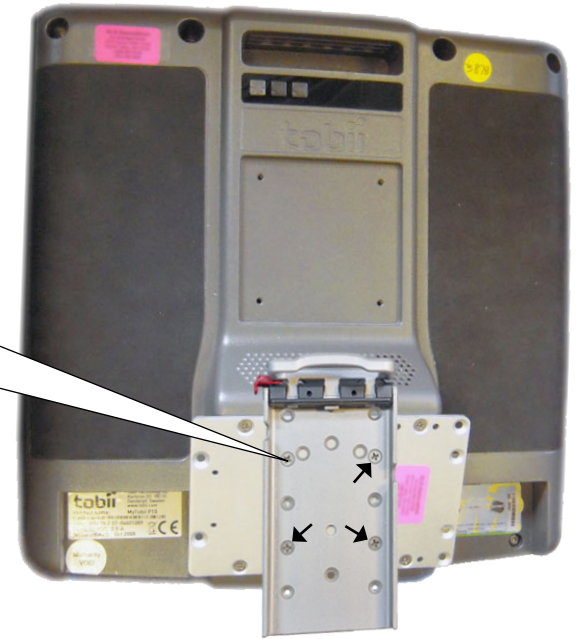
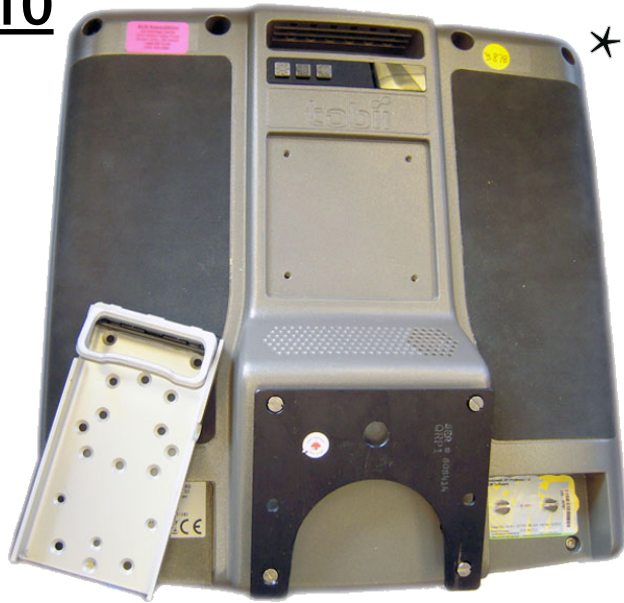


ATTACH DP-T1



ATTACH QRP

LightWriter SL40

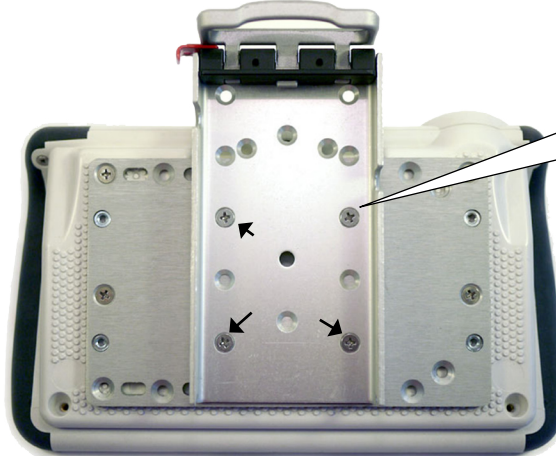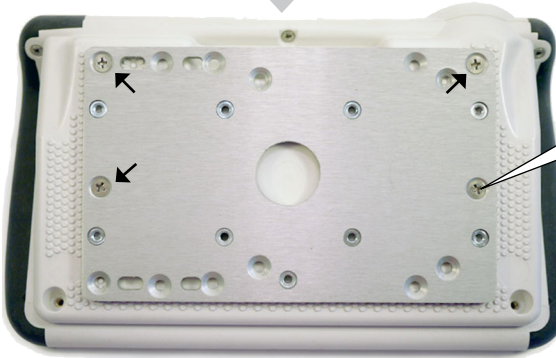

Attach Tobii device plate (DP-T1)

x4



M4-.7 x 10mm

Attach quick release plate (QRP)

x4



8-32 x 1/4"

mountnmover.com

S32

