

Using the Quick Release Plate (QRP): Attaching a QRP to a tray

All Bluesky Designs trays attach to the quick release plate (QRP) using 3 screws (included). Use the instructions below to attach the tray to the QRP.

- Place the flat side of the QRP flush with the attachment plate (back side of the tray).
- See the diagrams below to determine which hole spacing pattern will suit the users need.
- Align holes in QRP with holes in the attachment plate.
- Using the 3/18" Phillips screws provided, attach QRP to tray.

NOTE: Trays have 2 sets of attachment points!

