

Tilt'n Turner QUICK GUIDE



READ SAFETY TIPS

Maximum load = 15 pounds

Be sure mount is secure:


- Check bracket to wheelchair
- Check mount to post
- Check device to tilt plate

Do not attach to armrest, if it is unstable

Do not unlock/move mount when wheelchair is tilted

Put in locked position before driving or tilting wheelchair



Stay clear of pinch points

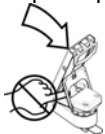

Adjust friction for specific user

For more info, read full manual
For support: 1-888-724-7002


Tilt'n Turner Parts

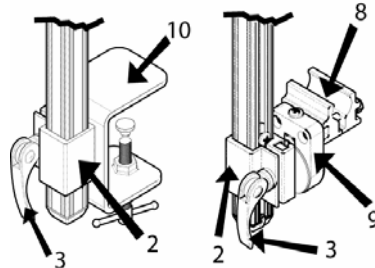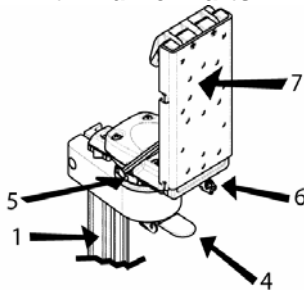


1. Post
2. Square Clamp
3. Cam Clamp
4. Paddle
5. Lock Setters
6. Tilt Lock Lever
7. Quick release Plate
8. Round Clamp
9. Wheelchair Bracket
10. Table Clamp

Post Adjustments

1. Unlock
2. Align ridge w/ gap

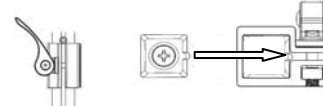

3. Insert Post


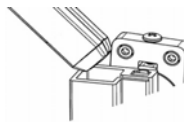


4. Lock



Height Setting

Loosen, adjust & tighten the height set screw

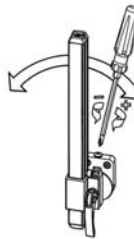
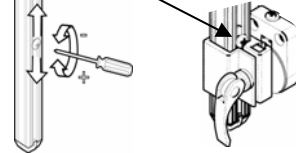


Setting post tilt angle:


Loosen, adjust & tighten tilt screw.

Quick Release Plate


Attaching and locking a quick release plate

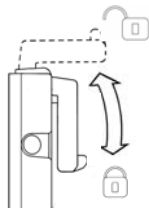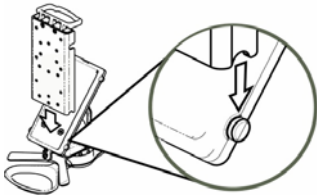


Tilt angle & lock adjusting the tilt

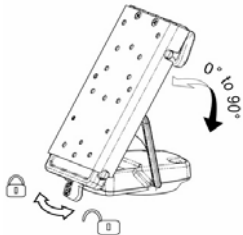

Moving the Tilt'n Turner

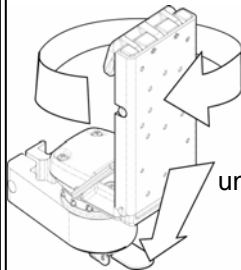

Paddle unlocks the joint

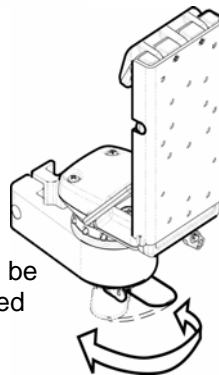


Paddle can be repositioned

For more info:
Refer to full manual
Contact BlueSky Designs
1.888.724.7002
www.blueskydesigns.us


Setting Lock Positions

Lock setters provide repeatable user-specific positions.

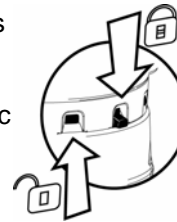

Lock position:

Lock setter is down and in line with friction adjustment screw

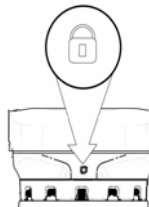

Bypass position:
Lock setter is

