


# Adapter Plate 5 (WC-AP5) Multi-Use Plate

Scale 1 : 1

1/4-28 threaded Hole (Qty = 4)

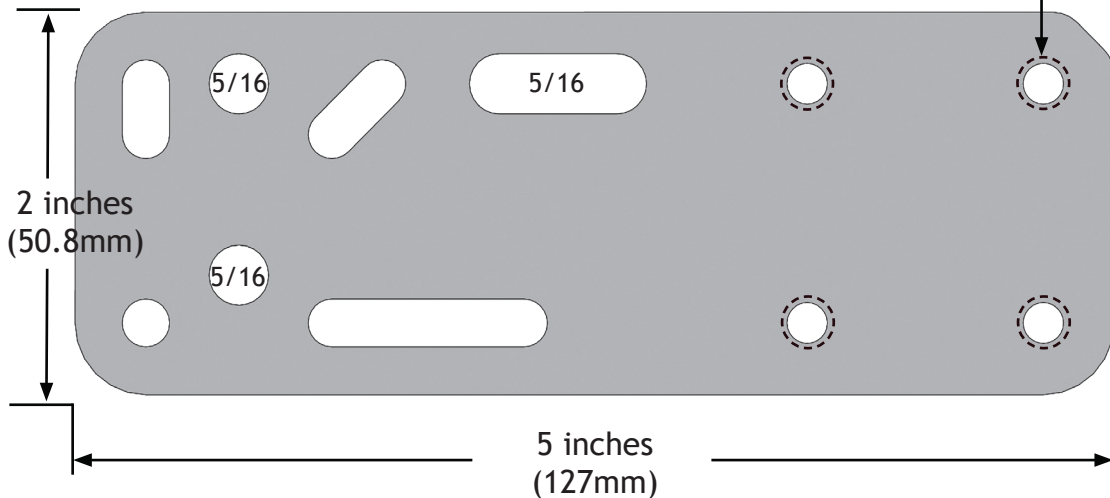

## Plate to Plate Attachment Options

- Non-threaded hole to threaded hole = Good
- Non-threaded hole to non-threaded hole with nut = Good
- **Threaded hole to threaded hole = Not good (will damage threads!)**

## Hole Pairing Options

- Attaching with two holes = **BEST**
- Hole and slot (horizontal or vertical) = Good
- Slot and slot (both horizontal) = Good
- Two vertical slots = **NOT GOOD!** (could slip!)

\* See Quick Guide and website for specifics

## Set Includes:

(1) Adapter Plate 5 (WC-AP5)

### Hardware - Plate to Plate Attachment

(2) 1/4-28 x 1/2" Socket head cap screws

### Hardware - Unitrack:

(4) M6-1 x 14 Socket head cap screws

(4) M6-1 x 10 Square nut

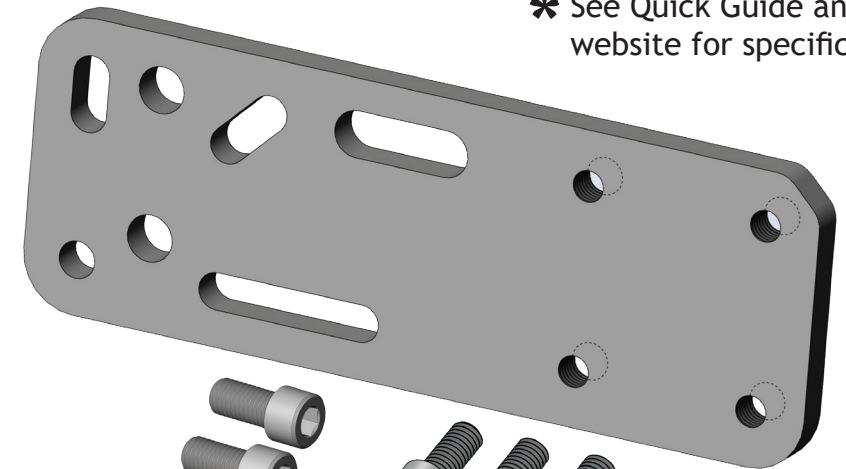


Plate to Plate Attachment

Permobil  
**UNITRACK**  
ONLY!

### WARNING!

Never attach two plates via threaded hole to threaded hole! This damages the threads and the connection can loosen!

**Mount'n Mover**™  
by BlueSky Designs

[mountnmover.com](http://mountnmover.com)

**Product:** WC-AP5, Adapter Plate 5

**Weight:** 10.6 oz. (0.3 kg)

1-888-724-7002 (toll free) or 1-612-724-7002

pg 1 of 2

# Wheelchair Attachment Overview

\* All plates are considered 'multi-use plates' and can be configured, combined, and used in a variety of ways.

