



Custom quickconnect Attachment

The custom quickconnect includes four attachment screws to attach to the quick release plate. Fasten the quickconnect to the top four countersunk holes in the quick release plate using the screws provided.

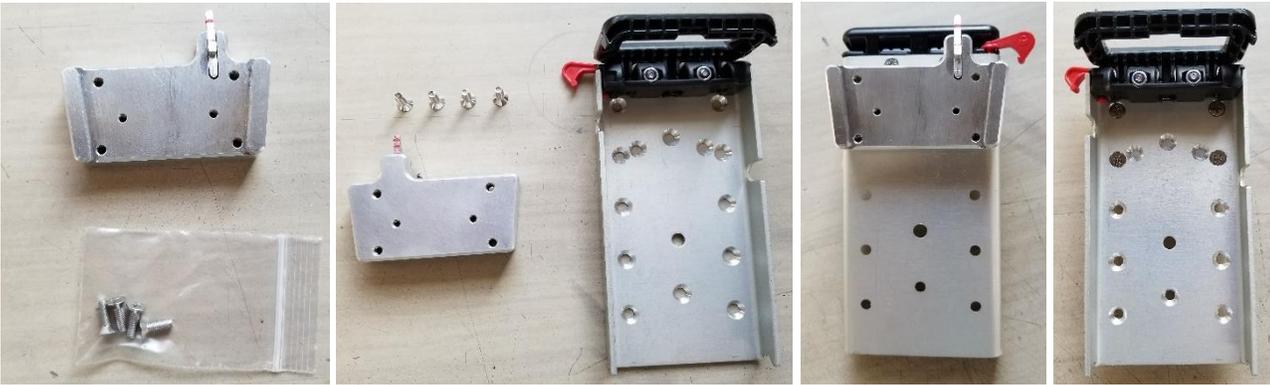

Attaching the Tobii Dynavox Indi to the quickconnect

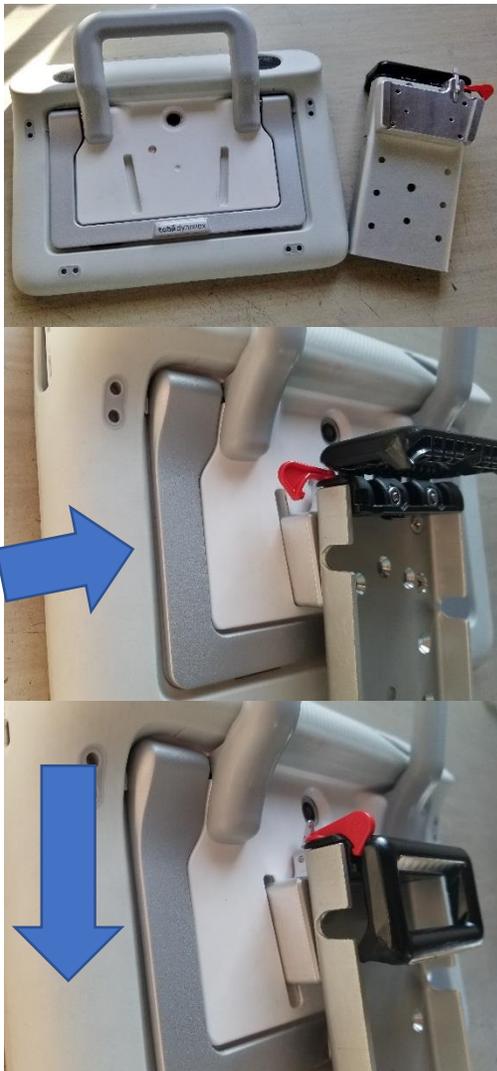

The Indi device (with mounting boot) has integrated mounting slots compatible with the quickconnect

Then slide the device down until you hear a click or see that the locking lever has engaged. Pull up on the device to check that it is secure.

To attach, position the device to engage the slots, ensuring that both slots are engaged.

Removing the Tobii Dynavox Indi from the quickconnect

Locate the release lever (circled in red)



While depressing the lever, pull up on the device for removal.

