

Simple Mount - Small and Large

Simple Mount Small

(3.5" x 7.0" x .25")



Simple Mount Large


(9.0" x 11.5" x .375")



***Holds 15 pounds!**

*Available in Post lengths: 8", 12", 18", 24", and 36"

Lip Supports

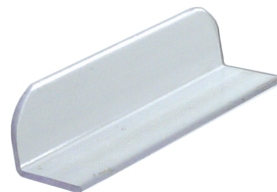

Simple Mount Small includes 1 Lip Support
1/2" tall x 2" long

Simple Mount Large includes 2 Lip Supports
1" tall x 2" long

**Additional Lip Supports sold separately*

CAUTION!

Do not loosen or remove bolt!



Max load
15 lbs!




Caution! Do not use as a lift support



Wheelchair Attachment

Insert the Simple Mount post into the Wheelchair Bracket (WB2)


See website for instructions on how to use the WB2

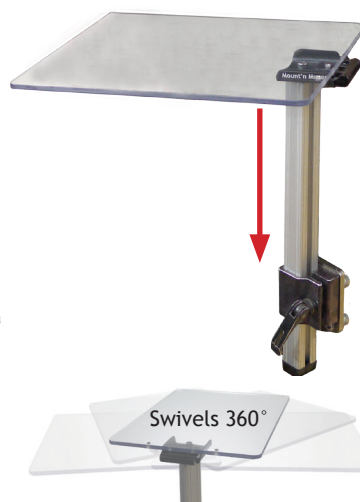

Wheelchair bracket is attached to a wheelchair using Mount'n Mover hardware (sold separately) See website




LAE

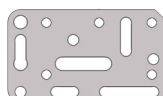

AP4




AP2

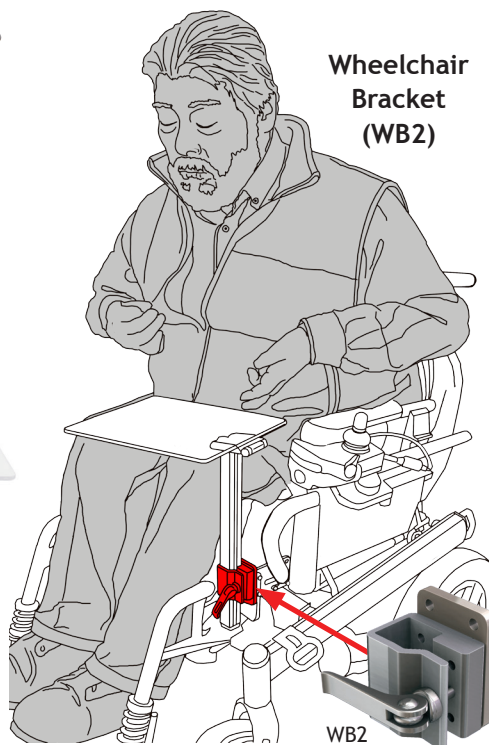

Wheelchair Bracket (WB2)

Offset Option

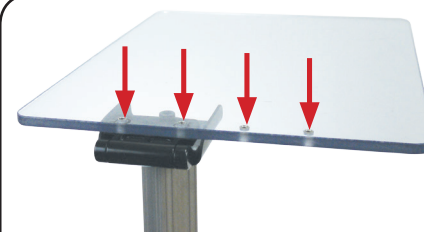

Offset the Simple Mount LG 1-3/8" to the right or the left




Offset left



Center



Offset right

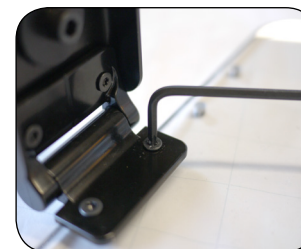


Use a #T15 Torx wrench (included) to remove the screws holding the tray to the hinge. Move the tray to the desired offset location and tighten the screws.

Lip Supports: Attaching



Wipe surface of the tray and bottom of the lip support with the included alcohol pad. This greatly increases the strength of the connection when attaching the lip support to the tray

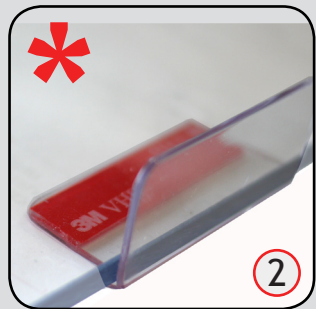


*** Important!!**

Place tape on furthest edge from bend of lip support. This provides the most strength!


Create a channel (may require additional Lip Supports, sold separately)




Attach two Lip Supports together to form a "channel" to hold your device

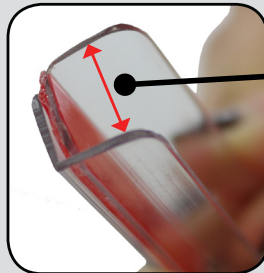


Large Lip Support channel has depth of 1 inch. Small Lip is 1/2 inch

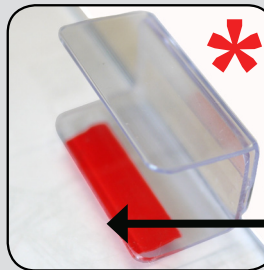

Showing channel configuration with the tape placed furthest away from bend on lip support

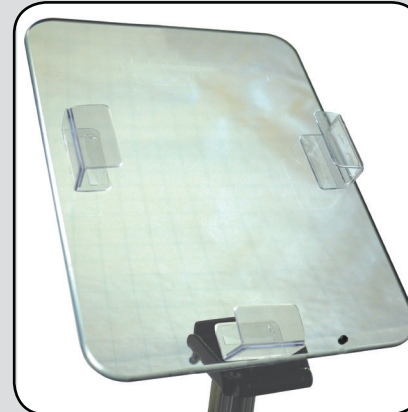

Example of Lip Support "channels" attached to the Large Simple Mount

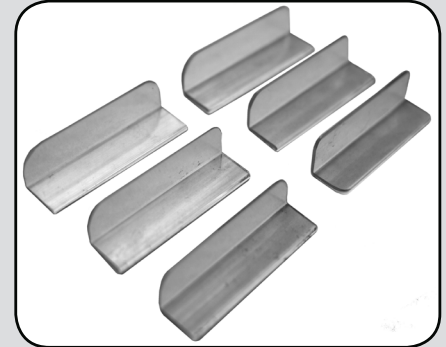

Additional Lip Supports Available:

#TRA-LS-1

Large Lip Supports (set of six - 1" tall x 2" long)


#TRA-LS-1/2

Small Lip Supports (set of six - 1/2" tall x 2" long)

- * Simple Mount Large includes two Lip Supports
- * Simple Mount Small includes one Lip Support

Simple Mount - Small and Large Additional accessories

Table Attachment

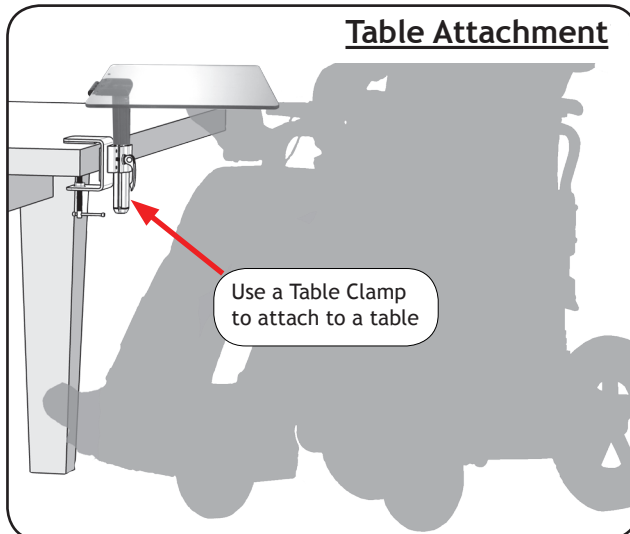

Use a Table Clamp to attach to a table

Floor Stand



Combine with a Floor Stand for a mobile surface

mountnmover.com
1-612-724-7002

Double Decker



Combine with a Single or Dual Arm to make a double decker

Link Cube

Remove the Link Cube from the Link Bracket. Place it in the desired position on the post and reattach the Link Cube to the Link Bracket