

Introducing the RedLock:



The **RedLock** was developed to provide a secondary lock to keep people from accidentally unlocking their device from the Mount'n Mover. Devices easily attach to and detach from the Mount'n Mover via the Quick Release Plate (QRP) -sometimes a little too easily.

Another benefit of the **RedLock** is that it provides a visual check to see whether the device is locked. When the **RedLock** is rotated inward, it positively snaps into place and prevents accidental unlocking of devices. The Quick Release Handle cannot be lifted, so the device cannot be removed.

A person needs to intentionally rotate the **RedLock** out of position to lift the Handle and remove the device.



The **RedLock** in-place.



Closeup of the **RedLock**

CAUTION:

Although the **RedLock** prevents accidental QRP removal, if the QRP is put on incorrectly – it may still come off even with the **RedLock** in place.

Always be sure to check that the QRP is properly attached!

How to use your RedLock:

1. When the RedLock is like shown, you can freely remove or attach the Quick Release Plate (QRP).



2. Turn the RedLock counter-clockwise to prevent the QRP handle from lifting.



3. With the RedLock in this position, you cannot accidentally remove the QRP.



4. To remove the QRP, turn the RedLock clockwise, allowing the QRP handle to lift and remove as normal.

